BATHTUB SEAT

Art. 302012

Gebrauchsanleitung
Brugervejledning
User Manual
Manuale
Bruksanvisning
Instrukcja Obsługi
Manuel de l'utilisateur

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1. Introduction

Thank you for buying a high-quality Mobilex product. These instructions of use contain a description and important information about the use of the medical device. It is intended to help you to use the product correctly and safely. It is very important to read the instructions carefully before using the device. Pay particular attention to the safety guidelines and always follow them. Mobilex is constantly trying to improve and upgrade its products. Therefore we reserve the right to change specifications and technical data of the product without further notice.

Intended use of the device

The bath seat is a medical device which intended use is to give seating support when people with functional limitations, persons in wheelchairs and other people use a bathtub. It can be adjusted to suit the majority of bathtub widths.

Indications

The bath seat is an overlay for regular bathtubs, indicated for use as a seating device by people with functional limitations, persons in wheelchairs and other people with limited mobility of the lumbar spine, knees, or hips, and for all those who have difficulties with seating down low and standing back up from bathtub's bottom while performing everyday personal hygiene. Using the seat for purposes other than those indicated above, is prohibited.

Contraindications

This product is not indicated for those patients who can't maintain proper body balance in a sitting and standing position. The device must not be used if the user has problems with impaired perception. Contraindications cannot be related to one specific clinical picture, and they depend on individual evaluation of potential user's health status. The choice of orthopedic aid appropriate for a given clinical picture must be always made with the help of a specialist physician or physiotherapist. In addition, observe the safety information in Chapter 2.

Areas of use

The product can be used in home care, health and long-term care facilities, and hospitals.

Medical incidents and unwanted events

In case of any unwanted events related to the device, please contact the dealer or manufacturer:

Mobilex A/S Grønlandsvej 5 DK-8660 Skanderborg Denmark	Phone: +45 87 93 22 20 Telefax: +45 87 93 17 77 Email: info@mobilex.dk
---	--

2. Safety requirements

- Make sure that this manual has been read by all persons using the device.
- The manufacturer assumes no responsibility for damage and/or injury resulting from failure to follow the operating instructions.
- Only use the product in a flawless condition.
- If defects or errors are detected you must immediately stop using the device and contact the dealer.
- Follow all manufacturer's recommendations and warnings.
- Be aware of the information on the product label.
- Only use the product for the described purpose.
- Don't stand on the device.
- Avoid making constructive changes to the device, unless you have the manufacturer's written acceptance for such modifications.
- The device must not be loaded with more weight than indicated in Technical data.
- The shower board must be precisely adjusted to the bathtub width.
- Always check the stability of the product before use.
- The shower board must be mounted so that it cannot move in the bath.
- The surface of the bath board must be well supported by the sides of the bath.
- Help of an assistant is still recommended for the use of the device.

Parts supplied

The device is supplied in a carton box. The contents of the package are:

- ✓ 1 Seating pad;
- ✓ 1 Seatframe;
- ✓ 1 User manual;

Preliminary check

Check the contents of the box with the list above. If at this stage you discover that a part is missing or damaged, contact your supplier immediately.

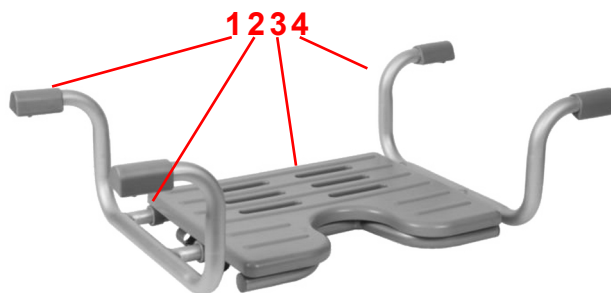
3. Description

Product description

The bath seat set consists of base frame made of aluminium and soft polyurethane seating pad and PU backrest (optional). The four arms of base frame are covered with protective rubber feet to be placed down on top of bathtub's walls.

The device consists of the following elements:

1. Rubber foot
2. Mounting slot
3. Soft seating pad
4. Support arm

Assembly description

Prior to use, put the soft seating pad on the frame by fitting 4 pins of the seat with slots in the frame.

Function description

Adjusting the support arm spacing

Adjust the position of all support arms so that the rubber feet will all reach the walls of the bath in a way that the device will not move from side to side when put on the bathtub. To adjust support arms on one side of the device, pull out 2 locking clips located under the arm's mounting slots. The arms can then be pulled out by pushing in the locking fetter and pulling the support arms. All arms should be adjusted to the same spacing.

General use

The bath seat can be easily placed on top of the bathtub's walls by putting the rubber feet on the walls of the bathtub.


Accessories

There are no accessories provided for this device.

4. Technical data

Bathtub seat	Art. 302012
Dimensions (W x D x H)	73,5 - 87,5 x 42 x 23 cm
Seat dimensions (W x D)	41 x 34 cm
Suitable for bathtubs	60 - 75 cm
Weight	2,60 kg
Max. User weight	150 kg

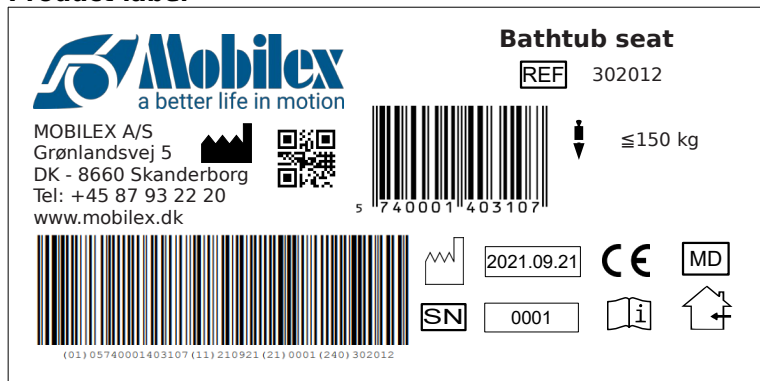Product label

Example of the product label

	QR-code link to homepage		Manufacturer
	CE marked product		Item Number
	Observe the user manual		Max load
	Medical Device		Manufacturing date
	To be used indoor and outdoor		Serial no

Meaning of the icons

5. Care and maintenance

Regular dirt can be removed with standard cleaning products and a sponge or soft rag. Check the specific product information and use only commercial cleaning products which are suitable for cleaning. For disinfection, use only disinfectants certified for use with medical devices available locally.

Despite the solid construction and the use of resistant materials the product is subject to wear. It is therefore recommended to check the product at regular intervals by a professional service. We recommend the following:

- ✓ Cleaning and disinfection of the product
- ✓ Checking the frame for bends, damage, wear or corrosion
- ✓ Checking wear parts and replacing them if necessary
- ✓ General re-tightening of the components

Recommended service life: Normally 10 years, depending on use.

Storage: We recommend storage in a dry place in temperatures above 0 °C.

Transportation

The device should be transported in a stabilized and secured way with no risk of sudden or dangerous movements during transport.

Reuse

The product is suitable for reuse. Before a new user takes over the product, the device must be checked on a technical level and disinfected according to point 5.

6. Most common problems and solutions

<i>Problems</i>	<i>Possible causes</i>	<i>Solutions</i>
Soft seat pad is not fixed in position (shaking or moving).	One or more of seat holding pins are not inserted correctly into the frame holes.	Take the bath seat up from the bathtub and turn it upside down to check position of seat holding pins. If necessary push the seat pad and press all pins into their holes on the main frame.
The bath seat doesn't lie down flat and stable on bathtub side walls.	Rubber arm supports are not in the same parallel and flat position.	Check if all arms are adjusted to the same width and if all 4 rubber supports are adjusted in the way, so their flat sides are in the bottom of the frame. Adjust the width and/or position of rubber supports if necessary.

7. Disposal of the product

The product can not be disposed of with household waste and must be brought to the local recycling center.

8. Guarantee terms

Mobilex A/S offers 2 years of warranty for damages caused by errors in production or material errors. The warranty is not valid by not-intended repair or usage of the product. Parts subject to normal wear are not covered by the warranty, unless wear is caused by a manufacturing fault. If product changes are made without our written acceptance the CE-marking and product warranty will not be valid. In the event of product damages covered by this warranty, please notify your dealer or Mobilex A/S directly. The warranty does not cover transport costs and does not include compensation for personal injuries or non-use of the product during repair. The warranty does not cover damages due to the user manual not being followed.

9. Declaration of conformity

The device has passed the conformity assessment in accordance with EU Regulation: MDR 2017/745 regarding medical devices and is CE marked. A declaration of conformity has been issued for the device and is available at:

mobilex.dk/medias/2021-07/ce_aqua_badekarsaede_aluminium_pu_saede_302012_8303.pdf

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